

Uniden®

SSE VR1 **Amplified Voice Stereo Receiver**

For more exciting new products please visit our website:
Australia: www.uniden.com.au
New Zealand: www.uniden.co.nz

OWNER'S MANUAL

IMPORTANT SAFETY INSTRUCTIONS

The SSE VR1 is an Assistive Listening Device. Please read this Owner's Manual in full for details on the installation and operation of the SSE VR1.

Basic safety precautions should always be followed to reduce the risk of fire, electric shock and injury to persons.

Some other things to help you make the most of your SSE VR1.

- **This unit is NOT waterproof.** Do not immerse the unit in water or expose it to rain or moisture.
- Use the charger indicated in this manual.
- Do not expose the unit to excessive heat, smoke, dust, chemicals and vibrations.
- Do not use this unit at the station or construction site. As it may be difficult to hear the background noise.

IMPORTANT INFORMATION FOR CARE PROVIDERS

Uniden's Assistive Listening Device ("ALD") is an amplification product that is designed and intended to assist persons with a hearing difficulty.


As the ALD can generate considerable additional volume, Uniden advises that it should not be used by persons who don't have a hearing difficulty, because such use could result in hearing damage or loss.

MEDICAL WARNING

The ALD may affect the operation of other medical devices that you wish to use at the same time, including (for example) a pacemaker or hearing aid.

Uniden advises that before using the ALD, you should verify with the manufacturer or supplier of any such medical device that you can use it safely in conjunction with the ALD.

WHAT'S IN THE BOX?

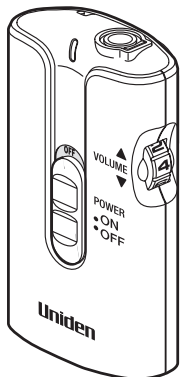

SSE VR1



Belt Clip




Earpiece Mic*




Additional Ear Caps


(Large,
Medium,
Small,
Extra Small)

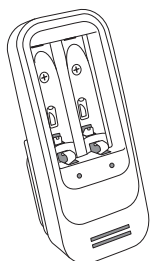

Charger

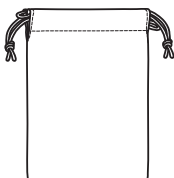

Carry Bag

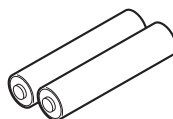

2x AAA Ni-MH Batteries**



Owner's Manual

* The earpiece mic comes with attached ear caps (medium size).





** The SSE VR1 needs only one battery to operate. Keep the spare battery for later use.

- If any item is missing or damaged, contact your place of purchase immediately. **Never use damaged products!**
- Need help? Get answers at our website:
www.uniden.com.au for Australian model
www.uniden.co.nz for New Zealand model.

CONTENTS

IMPORTANT SAFETY INSTRUCTIONS	2
IMPORTANT INFORMATION for care providers	2
WHAT'S IN THE BOX?	3
GETTING TO KNOW THE SSE VR1	5
GETTING STARTED	7
Battery Installation	7
OPERATION	8
Power On/Off	8
Adjust Volume	9
Volume Balance	9
Attach Belt Clip	10
MAINTENANCE	11
SOLVING PROBLEMS	12
GENERAL INFORMATION	13
Product Specification	13
Battery Information	13
Charger	14
Charging the Battery	15
ONE-YEAR LIMITED WARRANTY	16

GETTING TO KNOW THE SSE VR1

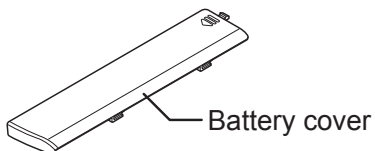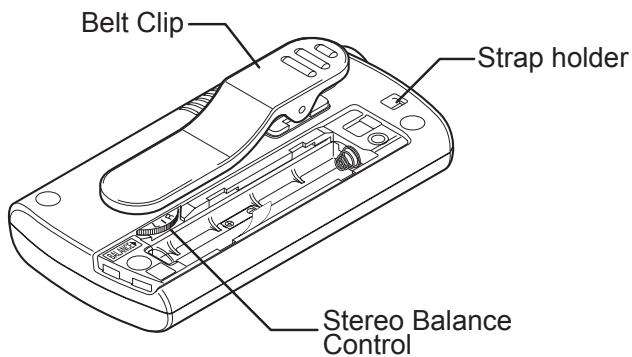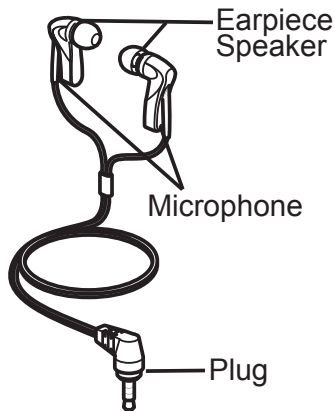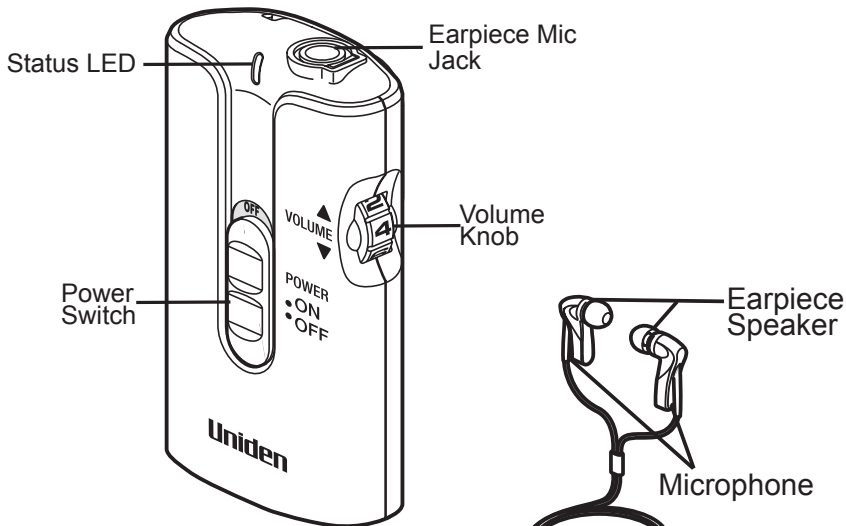

Buttons and Indicators	What it does
Status LED	Glows green when the SSE VR1 is turned on. For more information, refer to page 8.
Power Switch	Slide up to turn ON and slide down to turn OFF the unit.
Volume Knob	Rotate the knob to increase or decrease the volume.
Stereo Balance Control	Rotate the knob to balance the volume level between the right/left earpiece speaker.



GETTING STARTED

Battery Installation

Your SSE VR1 comes with 2 AAA rechargeable batteries to power the unit. The SSE VR1 needs only one battery to operate. Charge the battery completely (about 5 hours) before use (see Charging the Battery, p.15).



SSE VR1 can also operate with a AAA alkaline battery (not included).

Press in on the notch and slide the battery cover off.

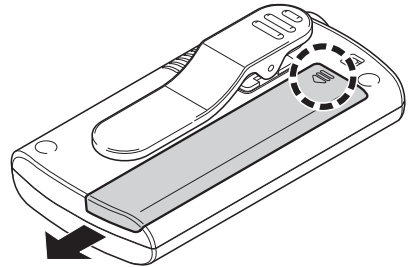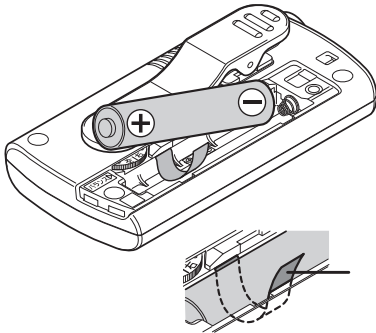

Install the battery in the battery compartment ensuring correct polarity. Ensure that the battery is placed over the ribbon.

When you want to remove the battery, gently pull the ribbon. This will release the battery from the groove.

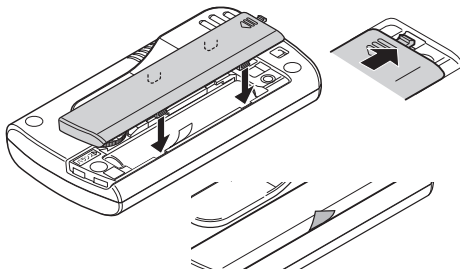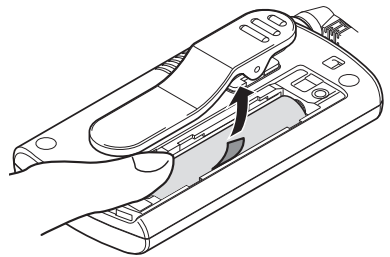


Align the battery cover over the slots provided and gently push it up. Please ensure that the ribbon stays inside.


OPERATION

Power On/Off

Insert the supplied earpiece mic into the earpiece jack.

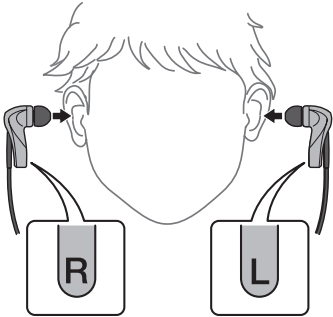


Put on the earphone (Look out for the 'R' (Right) and 'L' (Left) indication and ensure that you have got it correct).

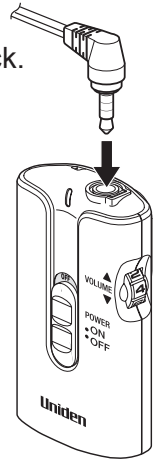

Slide the Power switch up to power on the unit. The Status LED will glow green indicating that the unit has been powered ON.


Slide the Power switch down to turn off the unit after use. The Status LED will turn off as well.

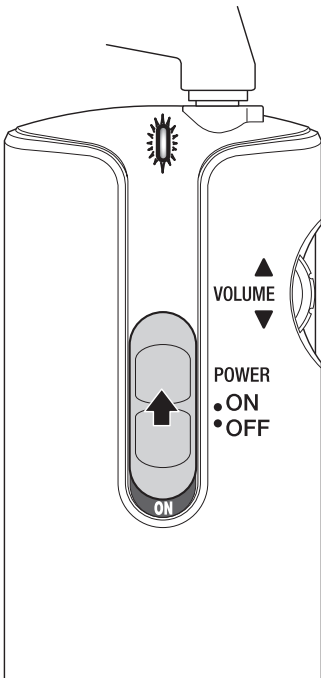

LED	What it Indicates
Green	Power On.
Green (Flashing)	Earpiece not connected.
Red (Flashing)	Low Battery
Off	Power Off. Battery empty


Adjust Volume

To adjust the volume on the unit, rotate the volume knob between (0 Min -10 Max).

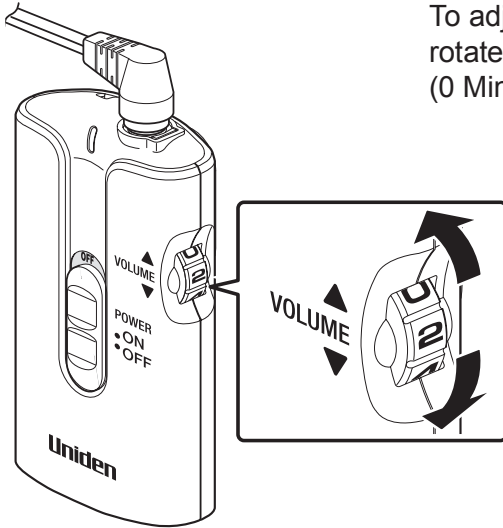



Volume Balance

You can use the Stereo Balance Control knob to adjust the volume between the right and left earpiece speaker. To reduce the volume on the left side rotate to L and to reduce the volume on the right side rotate to R.

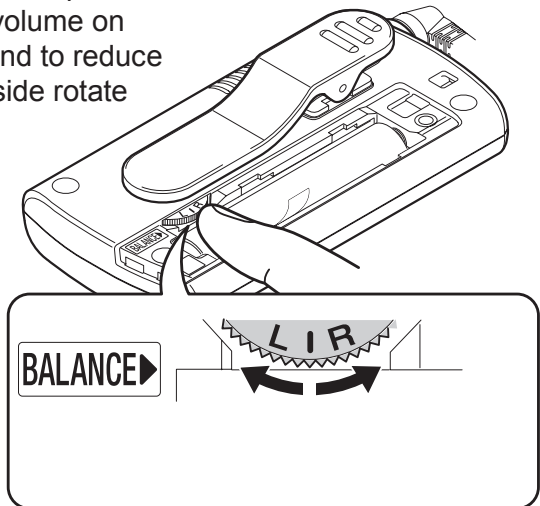


Attach Belt Clip

Align the protruding component of the belt clip (highlighted in the figure on the right) with the slot on back of the unit. Then gently press to fit and rotate the clip (as shown in the figure below) to secure position.

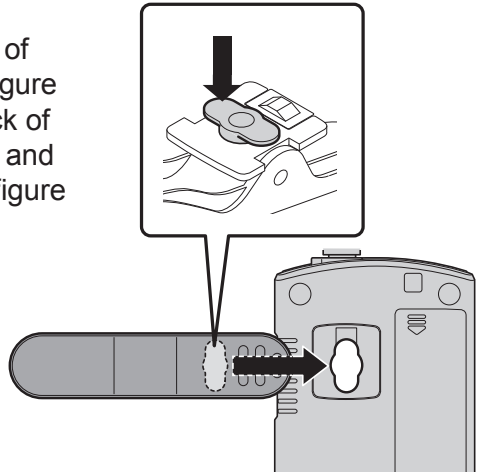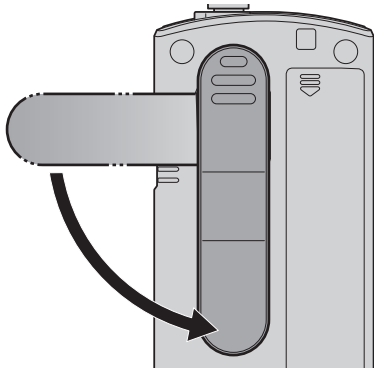

Remove the Belt Clip

Rotate the belt clip (as shown in the figure on the right) to release the clip from the slot on back of the unit.

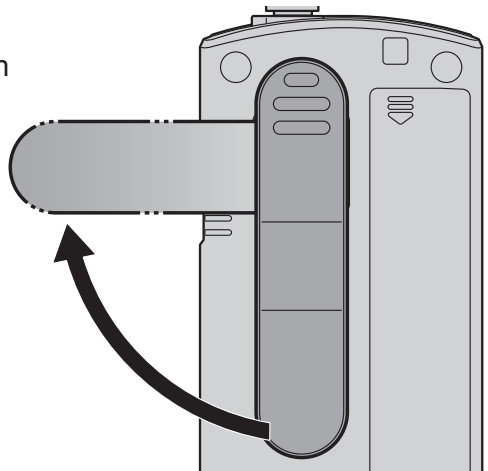

MAINTENANCE

- Use a clean dry cloth to clean the unit.
- Use a clean dry cloth to clean the earpiece mic.
- Clean the ear caps regularly - take off the ear caps and soak them in warm water to ensure there are no traces of dirt. Dry them thoroughly before using again.
- Never use scouring pads, abrasive cleaning liquids such as petrol or acetone to clean the unit.

SOLVING PROBLEMS

If you have any trouble with your SSE VR1, check this section first.
If you need help, visit our website listed on the front cover.

Problems	Possible Solutions
Unit won't power-on	<ul style="list-style-type: none">• Change to a charged battery.• Change to a new AAA alkaline (not included)• Check the battery direction.
I can't hear	<ul style="list-style-type: none">• Ensure the earpiece-mic is connected properly.• Please use earpiece-mic that comes with the unit.• Adjust the volume.
I can hear only from one side of earpiece speaker mic	<ul style="list-style-type: none">• Adjust L-R balance.• Please read through the Volume Balance section.
There's a lot of noise and the voice seems distorted.	<ul style="list-style-type: none">• Clean the ear caps.• Please read through the Maintenance section.
There's a howling sound.	<ul style="list-style-type: none">• Ensure both earpiece mics are used.• Reduce volume.• Ensure you've chosen the most suitable ear cap.
Red LED is blinking	<ul style="list-style-type: none">• Ensure the battery is charged.• Change to a new AAA alkaline battery (not included).
Green LED is blinking	<ul style="list-style-type: none">• Ensure earpiece-mic is connected properly.• Ensure you are using the earpiece-mic that came with the unit.

GENERAL INFORMATION

Product Specification

Model name	SSE VR1
Maximum audio output	110dB SPL
Maximum audio gain	40dB
Ear speaker type	Dynamic
Microphone type	Electret condenser type
Size	39mm (W) x 74.4mm (H) x 14.2mm(D)
Weight	25g (not including earpiece-mic, battery)
Operating Time	About 28 hours (NiMH) About 35 hours (Alkaline)
Operating Temperature	0°C to 40°C
Storage Temperature	-20°C to 50°C

Battery Information

Battery Type	NiMH AAA
Operation Temperature	0°C to 40°C
Storage Temperature	-20°C to 50°C
Capacity	800mAh, 1.2VDC


Charger

Operation Temperature	0°C to 40°C
Storage Temperature	-20°C to 50°C
Input Voltage	AC100-240V 50/60Hz
Output Voltage	1.4V DC
Charging current	220mA +/- 10%
LED Indication	2 x Amber Colour LED for two channels ; two channels operate individually
LED Status	
Power connected but without battery	LED Off
Charging in Progress	Slow blinking LED
Fully charged and switched to trickle charge	LED On
Bad / Primary battery	Fast blinking LED
Battery Type/Size	600 series ~ 1000 series (min. 600mAh) ~ (min. 930mAh)
Charging Time (Hours)	3.0 ~5.0


Charging the Battery

If you opt to use the AAA rechargeable batteries that come with the unit, then you could charge them regularly using the charger that also come with the unit.

Install the batteries (ensure the correct polarity) as shown in the image below.

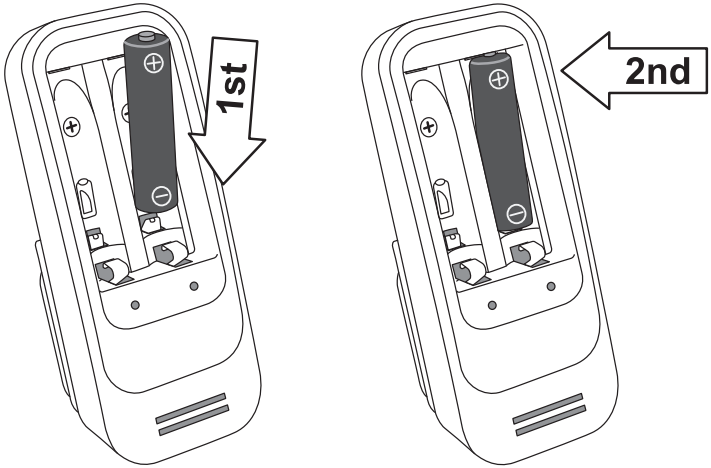

After charging the battery completely (about 5 hours), take it out as illustrated in the image below.

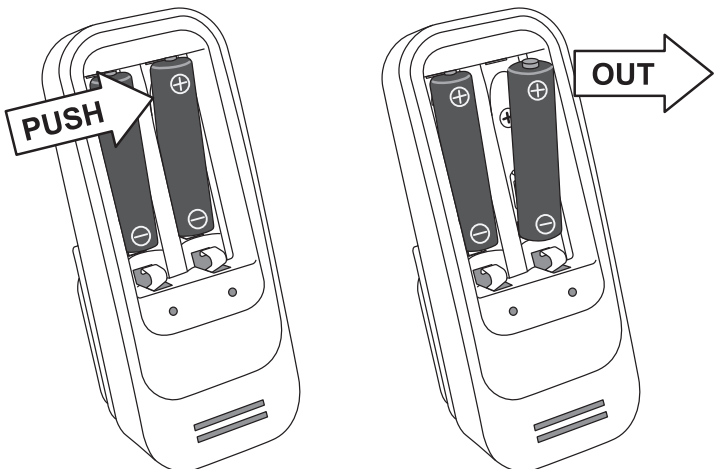

ONE-YEAR LIMITED WARRANTY

SSE VR1

IMPORTANT Satisfactory evidence of the original purchase is required for warranty service

Please refer to our Uniden website for any details or warranty durations offered in addition to those contained below.

Warrantor

The warrantor is either Uniden Australia Pty Limited ABN 58 001 865 498 (“Uniden Aust”) or Uniden New Zealand Limited (“Uniden NZ”) as the case may be.

Terms of Warranty

Uniden Aust/NZ warrants to the original retail purchaser only that the SSE VR1 (“the Product”), will be free from defects in materials and craftsmanship for the duration of the warranty period, subject to the limitations and exclusions set out below.

Warranty Period

This warranty to the original retail purchaser is only valid in the original country of purchase for a Product first purchased either in Australia or New Zealand and will expire one (1) year from the date of the original retail sale.

If a warranty claim is made, this warranty will not apply if the Product is found by Uniden to be:

- (A) Damaged or not maintained in a reasonable manner or as recommended in the relevant Uniden Owner's Manual;
- (B) Modified, altered or used as part of any conversion kits, subassemblies or any configurations not sold by Uniden Aust or Uniden NZ;
- (C) Improperly installed contrary to instructions contained in the relevant Owner's Manual
- (D) Repaired by someone other than an authorized Uniden Repair Agent in relation to a defect or malfunction covered by this warranty; or
- (E) Used in conjunction with any equipment, parts or a system not manufactured by Uniden.

Parts Covered

This warranty covers the Product and included accessories.

User-generated Data

This warranty does not cover any claimed loss of or damage to user-generated data (including but without limitation phone numbers, addresses and images) that may be stored on your Product.

Statement of Remedy

If the Product is found not to conform to this warranty as stated above, the Warrantor, at its discretion, will either repair the defect or replace the Product without any charge for parts or service. This warranty does not include any reimbursement or payment of any consequential damages claimed to arise from a Product's failure to comply with the warranty.

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.

This warranty is in addition to and sits alongside your rights under either the COMPETITION AND CONSUMER ACT 2010 (Australia) or the CONSUMER GUARANTEES ACT (New Zealand) as the case may be, none of which can be excluded.

Procedure for Obtaining Warranty Service

Depending on the country in which the Product was first purchased, if you believe that your Product does not conform with this warranty, you should deliver the Product, together with satisfactory evidence of your original purchase (such as a legible copy of the sales docket) to Uniden at the addresses shown below. You should contact Uniden regarding any compensation that may be payable for your expenses incurred in making a warranty claim. Prior to delivery, we recommend that you make a backup copy of any phone numbers, images or other data stored on your Product, in case it is lost or damaged during warranty service.

UNIDEN AUSTRALIA PTY LTD

Service Division
345 Princes Highway,
Rockdale, NSW 2216
Phone number: 1300 366 895
Email: custservice@uniden.com.au

UNIDEN NEW ZEALAND LTD

Service Division
150 Harris Road, East Tamaki
Auckland 2013
Phone number: (09) 273 8377
Email: service@uniden.co.nz

Uniden®